
2953 WHEELCHAIR ACCESS 29″ x 53″		Available Therapy Systems 2953WCA**SNon-Jetted Soaking 2953WCA**HWhirlpool Jetted 2953WCA**AAir Jetted 2953WCA**DWhirlpool & Air Jetted
<b>Key Attributes</b> Walk-in Type:	Wheelchair Accessible	<b>Key Measurements (inches)</b> Overall Outer:52.25 x 29.375 x 42.5
Material:		Interior Width:
Soaker Weight:		Standing Area:
Jetted Weight:		Seat Width:
Average Fill Capacity*:		Seat Height:
*Estimated with occupant inside. May vary based on occupant's volume.		Soaking Depth:
	-,	Drain to Side Wall:
Required Electrical	Service 110v	Drain to Back Wall:14
Air Bath or Whirlpool		Step-in Height:6.25
Whirlpool w/ Inline Heater	-	Door Clearance:n/a
Whirlpool & Air Jetted 1x 20 amp GFCI		Faucet Deck:
Soaker	-	Tile Flange:1
		Extension Kit:7

## It is vital that tub is completely level with all leveling feet touching the floor for the door, seal and drain to work properly.

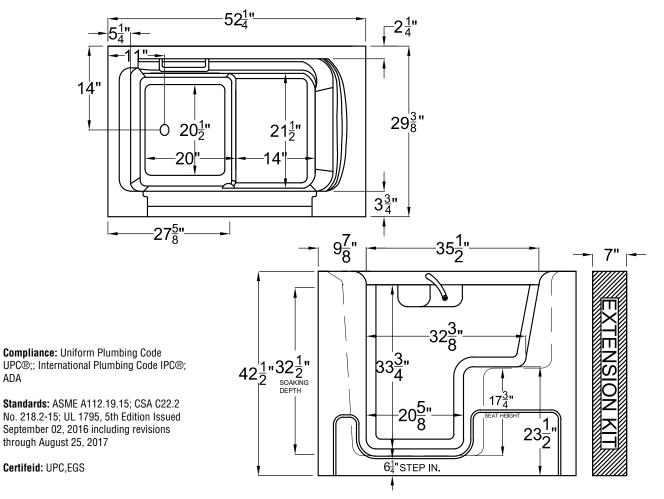

- IMPORTANT! TEST THE TUB'S DRAIN AND JETS (if any) BEFORE PERMANENT INSTALLATION!
- **INSPECT THE TUB WHEN RECEIVED!** Report any damages or missing parts within 5 business days of delivery date.
- Before ordering or installing, please consider future access to equipment.
- Please measure tub on-site before building tub surrounding!
- Jet, pump & blower locations are predetermined and cannot be changed.
- Tub door must remain closed when not in use.

- Please note that installer(s) must be certified or warranty will be voided.
- Measurements are subject to a +/-.5'' variance and subject to change without notice. Measurements here in supersede all other published specs prior to publication date shown below.
- Inline heater (if included) is designed to maintain water temperature during operation. It will not heat cold water to desired bathing temperature.
- Product must be registered for warranty activation. See user manual or visit SWCORP.com/register for more information.

## ailable Therapy Systems